

essential™



Cost Effective Restorations

At DTS not only do we try to bring new and innovative restorations and techniques to our customers, we are also committed to providing value for money.

Our essential™ range for crown and bridges provide a cost effective restoration for non precious bonded crowns and bridges. Non precious metal frameworks are made for each unit and waxed up in the traditional way prior to them being cast in the required alloy.

Indications:

- Crowns
- Bridges
- Maryland Wing
- Maryland Pontic

Advantages:

- Aesthetic and natural appearance
- Every crown comes with the DTS Statement of Manufacture
- 10 working day turnaround

Disadvantages:

Not recommended for patients whose teeth do not allow ideal preparation to support porcelain, teeth with short clinical crowns, edge-to-edge occlusion or whose opposing teeth occlude on the cervical fifth of the palatal surface.



Cost Effective Restorations


Product Facts

Impression Technique	Traditional
Material	Non -precious bonded
Indications	Single unit crowns, multi unit bridges
Recommended use	Full mouth
Margins	Feather edge, chamfer or shoulder
Cementation	Conventional cementation
Turnaround Time	10 Lab Days
Method of manufacture	Traditionally-cast & built up
Aesthetics	✓✓✓
Shades / colouring	Internal effects customised shading multi-layered
Strength	✓✓✓✓
Add-on options	Fit to denture, attachments etc. Charges apply
Benefits	Traditional method, multi-layer build
Warranty	1 year
Type	Standard & NHS
Popularity	✓✓✓


Help your patients understand your recommended restoration with our patient friendly cards.

Patient Support Literature


For more information or to order literature please call 0141 556 5619 or email restorations@dts-international.com

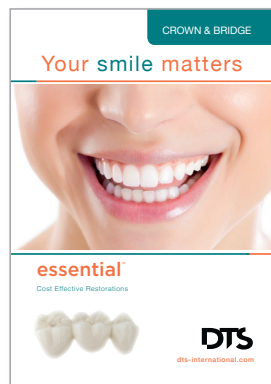

Preparation Guide

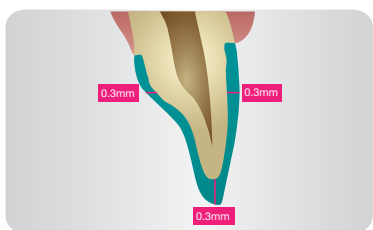

Recommended preparation for anterior teeth

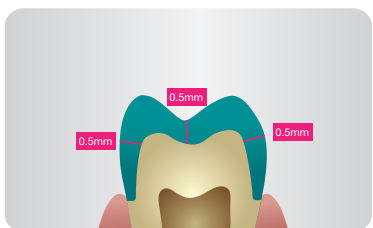

Recommended preparation for posterior teeth

What to use to attach to natural tooth:

We recommend the new crown be fixed in place with special dental cement or adhesive. The cement forms a seal to hold the crown in place.

How to adjust if required:

You can work on it just like you work on real tooth structure. This means you can adjust a crown for proper occlusion with additive or subtractive technique. No need to send it out.

Conclusion:

With an aesthetic and natural appearance the essential™ range provides fantastic value for money on a budget case.

DIGITAL RESTORATIONS

PROSTHETICS

IMPLANTS

CROWN & BRIDGE

